
601785
2.0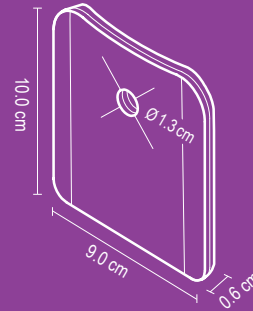

REF 907-A

PZN: 02918771

TRACOE care **Trachealkompressen**

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TRACOE care **Tracheal Compresses**

two-ply, without slit

TRACOE®
care

CE

MD

Medical
Device

Do not
Reuse

Store in a
Dry Place

Protect from
Sunlight

10 pcs.

Packaging
Content

Manufacturer

TRACOE medical GmbH
Reichelsheimer Str. 1 / 3
55268 Nieder-Olm / Germany
Tel.: +49 6136 9169-0
Fax: +49 6136 9169-200
info@tracoe.com
www.tracoe.com

DE TRACOE *care* Trachealkompressen

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EN TRACOE *care* Tracheal Compresses

two-ply, without slit, for neonates, children, adolescents and adults

TRACOE *care* Tracheal Compresses are used to provide padding between the tracheostomy tube and the tracheostoma. The compresses help to absorb secretions and protect the skin beneath the neck flange.

The curved edge of the compress is oriented towards the chin. The compress can be applied/removed as part of a tube change or independently by making a cut as far as the round opening. When cutting the compress, take care that no remnants of material end up in the stoma.

Change the compress at least once a day or several times a day, if secretion is heavy.

Warning: Using the compress for a long period of time or re-using it may give rise to a rash. In the event of an allergic reaction such as itching, reddening or other signs of hypersensitivity, remove the compress and consult a doctor. Avoid wrinkling the compress when applying it. When applying or removing the compress, make sure that the tracheostomy tube does not slip out of place.

When the compresses are used together with other medical devices, follow the respective instructions for use.

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