TRACOE tube clean Reinigungspulver

TRACOE tube clean Cleaning Powder

REF 932

DE	Gebrauchsanweisung
EN	Instructions for Use
FR	Instructions d'utilisation
ES	Instrucciones de uso
DA	Brugsanvisning
CS	Návod k použití

DE / Gefahr

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EN / Danger

Causes serious eye damage / Causes skin irritation/ May cause respiratory irritation / Wear protective gloves and eye/face protection / If skin irritation or rash occurs: Get medical advice.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Store in a well-ventilated place. Keep container tight closed / Dispose of contents/container to hazardous or special waste collection point.

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Instructions for Use for TRACOE tube clean Cleaning Powder

TRACOE tube clean Cleaning Powder has a capacity of 400 g. It is indicated for cleaning of tracheostomy tubes and place-holders.

It is composed of phosphate (> 30 %), complexing agents (< 5 %), nonionic surfactants (< 5 %), colorants and auxiliaries.

1. Warnings

Hazardous and safety precautions:

H318 Causes serious eye Damage

H315 Causes skin irritation

H335 May cause respiratory irritation

P280 wear protective gloves and eye/face protection

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P403+P233 Store in a well-ventilated place. Keep container tight closed

P501 Dispose of contents/container to hazardous or special waste collection point.

Always dissolve the TRACOE tube clean cleaning powder in water before use.

Do not swallow the dissolved cleaning powder.

Keep it out of the reach of children.

2. Functional Description

When the product is used together with other medical devices, follow their respective instructions for use.

1. Put a measuring spoon of cleaning powder (approx. 5 g) into a cleaning tub (e.g. TRACOE tube clean Cleaning Tub (REF 934)).
2. Solve the powder with lukewarm water (about 500 ml).
3. Submerge the tracheostomy tube or placeholder to be cleaned in the cleaning solution.
4. Leave the device in the solution for about 10 to max. 15 minutes.

5. Remove the cleaned device from the solution.
6. If needed, subsequent cleaning steps for the device can be carried out according to their respective instructions for use.
7. Afterwards, rinse the cleaned device thoroughly with potable or distilled water and dry them with a fluff free cloth.

The prepared cleaning solution is single patient only and will remain effective for 24 hours and should then be disposed of.

3. Storage

Keep the product within a clean and dry location, and away from sunlight.

4. Disposal

Used products are to be disposed of in accordance with valid national regulations, waste management plans, or clinical procedures governing biohazardous waste materials.

5. Returns and Complaints

Returned products, that have been used, will only be accepted, if TRACOE has agreed to the return and a completed decontamination certificate and complaint report is enclosed. These

forms are available either directly from TRACOE medical, or via the website www.tracoe.com.

EN

If the device is involved in a reportable incident, as defined in local medical device legislation, please contact TRACOE medical (complaints@tracoe.com), and the appropriate regulatory body in the country of use.

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TRACOE medical GmbH