**REF 940-D**

PZN: 02750981

### TRACOE tube clean Reinigungsbürsten

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### TRACOE tube clean Cleaning Brushes

straight, extra-long, suitable for tubes with inner diameter 11-12 mm

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## **EN** TRACOE tube clean Cleaning Brushes

straight, extra-long, for tubes with an inner diameter of 11-12 mm

TRACOE tube clean Cleaning Brushes are especially designed for cleaning tracheostomy tubes and placeholders. Flexible tubes should be straightened before cleaning to better pass through the curve of the tube with the brush. For rigid tubes, the brush should be bent to the shape of the tube before use.

The brushes are made of stainless-steel wire with nylon bristles.

After use, clean the brush thoroughly and hang it up to dry. For reasons of hygiene, the brush must be changed at least once a week.

**Warning:** Always remove the tube from the tracheostoma before cleaning. Never use the brush on the tracheostoma itself. It is best to introduce the brush from the tip of the tube. When the brushes are used together with other medical devices, follow the respective instructions for use.

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